

CURE PLUS 3/CURE MAX OPERATING INSTRUCTIONS

IMPORTANT: DO NOT USE WATER

1

Press and hold the power key for 3 seconds to turn on the controller.

Screen will display current temperature value.

Press "up" to see the set temperature value.

Press "down" to see the set temperature difference value.

Press and hold the "S" button for more than 3 seconds to scroll through Function Parameters "F1" to "F5" using "up" or "down" button.

Each menu item indicates the following:

Code	Function	Set Range	Default
F1	Temperature Set Value	-50° C - 99° C	10° C
F2	Difference Set Value	1° C - 10° C	3° C
F3	Compressor delay time	1 - 10 minutes	3 minutes
F4	Temperature Calibration Value	-10° C - 10° C	0° C
F5	Fahrenheit / Celsius Switching		0 / 1

2

To set desired temperature, press and hold the "S" button for more than 3 seconds to enter the modified parameter mode. It should display "F1" of the first menu item. If no button is pressed in 3 seconds after F1 is displayed, it exits modified parameter mode.

To adjust respective parameter reading, hold down the "S" key and press and hold "up" or "down" key for current menu item.

To change the menu item (F1, F2,...F5) for setting the parameter, press "up" or "down" key. Once you are at the menu item you want to set, hold "S" key and press and hold "up" or "down" key to set the parameter for that menu item.

4

Once all the parameters are adjusted, press "power" button to save the parameters and it should turn on cooling or heating depending on the parameter.

Indicator light should turn on display, to indicate cooling or heating.

To turn off controller, press and hold power key for 3 seconds to turn controller off.

Indicator Light	Function	Note
Cool Indicator Light	ON: Refrigeration starts; OFF: Refrigeration stops; FLASHING: Compressor delay	Cooling and Heat indicator light cannot be on simultaneously
Heat Indicator Light	ON: Heating starts; OFF: Heating stops;	
Set Indicator Light	ON: Parameter setting status	

Panel Instruction

